

STEP-BY-STEP GUIDE

How to Start Using the Evelyn Mobile App

Step 1

Search “Evelyn Mobile” in your mobile phone’s app store.

Once you download it, click “Sign Up” and enter your information to get started. Close the app and look out for a reset password message in the email you provided during registration.

Step 2

In the email, click the “Reset password” link.

Create and confirm your personal password. For added security, use a mix of letters, numbers, and special characters. Then press the “Sign In” button.

Step 3

On the main screen, click on the measurement you want to take, and it will open device usage instructions . You should be able to view the instructions manual directly within the app.

To see a real example of using the selected device, switch to the video tab & press play.

You can also view the history of your vital signs from your device, by scrolling to the “All Health Data” section.

Other benefits of Evelyn Mobile app:



Info section for your medication schedule & device user manuals.



Activity section to manage device notifications, view device readings & get usage instructions.



Interact section for our contact info & to interact with us.

To view the detailed Evelyn Mobile app step-by-step guide, visit accuhealth.tech/onboarding.